---

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This page left blank.

## UK

### Check list for annual service, Flex Integral BC/FC/NTC

All parts marked with  $\square$  are included in the service kit (SD-457) for the BC/FC/NTC.

All parts marked with \* should be available during your customer visit.

When we mention lubrication with grease you should use Flex silicone grease (YR-002 \*).

#### Cuspidor/suction/drain/amalgam separator

- Check and, if necessary, replace the internal waste water tubes and clean the drain junction.
- Replace the water-trap (AC-317 \*), if necessary (if there is one).
- Check and clean the drain pump (if there is one).
- Dismantle and clean the ejector (if there is one). Replace the nozzle, gasket, ejector tube, and the Filter, if necessary.
- Replace the external suction tubes, if necessary,  $\varnothing$  11 mm (AC-280 \*), and  $\varnothing$  16 mm (AC-279 \*).
- Dismantle and clean the Select Vac suction. Replace the membranes (AE-124  $\square$ ).
- Replace the Select Vac filter (AC-106 \*), if necessary, and grease the O-rings.
- Check and test amalgam separator, the separation automat, and the flushing unit (if there is one).
- Does the amalgam separator/ the separation automat work properly when sucking water from a bucket?
- Check the HygiFlex Vac function and the disinfectant pump, and clean the mixing tank (if a HygiFlex Vac is installed).
- Check the HygiFlex Vac Ultra function and clean the funnel and the container (if a HygiFlex Vac Ultra installed).
- Check if there are any leaks at the cup filler tube and the cuspidor flush tube. Replace the O-rings (UC-051 \*), (UC-539 \*), and (MR-209 \*), if necessary, and grease them.
- Remove the cup filler bowl, and grease the O-ring (MR-077 \*).
- Check and adjust the water flow of the cup filler, if necessary.
- Remove the cuspidor bowl, and grease the O-ring (MR-214 \*).
- Replace the gold trap (MR-075 \*) or the complete filter cartridge (SD-401 \*).
- Replace the gasket at the plungers (SG-233  $\square$ ) of the solenoid valves of the cup filler, the cuspidor Flush and the HygiFlex Vac/Ultra.

# XO Flex Integral, all models

## Annual Service kit SD-457

---

### Unit Stand

- Clean the coarse filter in the water supply inlet. Replace the filter cartridge (SD-232 ☞) and the O-ring (SD-259 ☞), and grease them. In case of cracks in the filter glass replace the complete filter (AC-168 \*).
- Clean the water reduction valve, replace the membrane set (SD-249 ☞), and grease it.
- Clean the air filter. Replace the filter cartridge (SD-237 ☞), and the membrane set (SD-233 ☞). Check the steam trap.
- Check the main air pressure (5.5 bar) and the main water pressure (2.0).
- Check and replace the internal connection hoses.
- Check the main solenoid water valve, and replace the gasket (SG-233 ☞) of the plunger.
- Check the electrical connections and the fuses on the MAIN CONTROL.
- Only FC/NTC:** Check the HygiFlex Thermo boiler (if fitted). De-lime the heating element (while it is de-mounted). You must not let any de-liming agent flow into the water system of the unit. Replace the O-ring (UC-105 \*) on the heating flange if necessary.
- Check the temperature and the function afterwards (the process lasts approx. 15 - 20 min.)
- Only FC/NTC:** Replace the lime filter cartridge (SD-232 \*) after the boiler (if there is one).

### Instrument Bridge

- Check the attachment of the bridge to the arm. Mount the fastening kit (AC-599 \*), if necessary.
- Check the electrical connections and fuses on the INSTRUMENT CONTROL.
- Only FC/NTC:** Check and, if necessary, replace the water filter (UC-034 ☞) in the valve block. If The HygiFlex Thermo is mounted it must be replaced with (MR-135 \*).
- Only FC/NTC:** Check and clean the nozzle between the valve block and the water filter block. Necessary, replace the nozzle (AC-114 \*) or the 3 O-rings (UC-162 \*).
- Check and, if necessary, replace the instrument tubes.
- Replace the O-rings (SC-740 ☞) of the Quick Flex couplings, and grease the thread and the O-rings Using Lubrimed grease (SD-318 \*) (otherwise used for the turbines).
- Check the select valves for function and leakages. Replace if necessary with an exchange valve (ZC-568\*).
- Check the turbine driving air according to specifications. **Note:** Now three turbine types with different specifications. Consult your Technical Service.
- Check the motor. If there is carbon dust in the motor, blow it free. Check the cooling air (8 l/min.) And replace the O-rings (SA-024 ☞) on the coupling. Return the motor for repair, if necessary.
- Check the water flow to the turbine, the motor, and the scalar. Can they be adjusted properly?
- Check the multi-function syringe and adjust the water flow (min. 75 ml/min.) and the air flow, if

# XO Flex Integral, all models

## Annual Service kit SD-457

necessary. Replace the O-rings (UC-726 ☞). Grease the O-rings with Paraliq (YR-032 \*).

- Cleanse the oil drip pan and replace the sponge (MC-459 ☞).
- Only FC/NTC:** Replace the gasket of the plunger (MR-150 ☞) in the solenoid water valve (AC-163 \*).
- Only BC:** Replace the plunger (SA-555 \*), the spring (SA-556 \*), and the O-rings (SD-303 \*), (SD-307 \*) and (SD-308 \*) in the solenoid water valve.
- Only screen with backlight:** Adjust contrast and light if necessary.

### Miscellaneous

- Check the adjustment of the arm systems (balance and friction), and adjust if necessary.
- Check the foot control and all the functions of the unit and the instrument. Also HygiFlex Thermo if mounted.
- Check the foot control cable for visible damage.
- Clean the rubber and the pedals with benzene, or replace the torn parts if necessary.
- Repair any damage to the enamel.
- Clean and dry the surface of the unit.
- Check to see that the dentist has a sufficient stock of spare parts (such as bulbs, grease, O-rings etc.).
- Check the function of the patient chair and adjust the brake and the lock system of the headrest, if necessary.

### Special tools and other materials for the service:

#### Tools

AC-188 Service kit, antistatic  
UC-600 Hand strap for AC-188  
SD-240 Manometer QF 4+4  
FA-400 Flowmeter, cooling air  
SD-391 Key, upper part, reduction valve  
SD-392 Key, bottom valve, reduction valve  
SD-393 Key, receptacle, air filter  
SD-388 Key, adjustment headrest  
AC-892 Test flow meter, water  
SA-007 Key, distance piece, turbine type P  
SA-010 Key, spray cover, turbine type P  
SA-011 Key, cap, turbine type P  
SA-983 Tool, spray cover nut, turbine type B  
SA-984 Tool, spray cover, turbine type B  
AE-308 Trimming pin set for screen with backlight, 2 pcs.

#### Material

YR-002 Silicone grease  
YR-032 Syringe with Paraliq grease  
YR-031 Tri-Flow lubricant  
YR-001 Flex Make Up 150 ml  
UC-342 Pressol grease 40 g  
YR-037 Repair paint, 9018  
YR-038 Repair paint, grey  
YR-039 Repair paint, white  
YR-040 Repair paint, blue  
YR-041 Repair paint, green  
YR-003 Repair paint, red  
SD-318 Lubrimed grease

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Tel +45 7020 5511  
Fax +45 7020 5510  
info@xo-care.com  
www.xo-care.com

This page left blank.

XO Flex Integral, all models  
Annual Service kit SD-457

Parts contained in Annual service kit SD-457		
Item	Qty	Description
AC-632	1	FILTER WITH RIM,HYGIFL. THERMO
AE-124	2	HOLDER WITH BELLOWS, SELECTVAC
MC-459	1	SPONGE, OIL WELL
MR-150	3	GASKET, SEAT, SELECT VALVE
SA-024	6	O-RING, BIEN AIR/FLEX MOTOR
SC-740	6	O-RING, UNIFIX COUPLING, BIG
SD-232	1	FILTER, WATER
SD-233	1	MEMBRANE SET, AIR
SD-237	1	FILTER, AIR
SD-249	1	MEMBRANE SET, WATER
SD-259	1	O-RING, FILTER HOUSING
SG-233	4	GASKET,SEAT,VALVE SD-292
UC-726	2	O-RING Ø3X1, SILICONE, 60 SH.
YA-695	1	Check list, annual service XO Flex Integral.

As an authorized XO Technician I hereby confirm, that annual service has been performed at your XO Equipment according to the manufactures procedures and instructions.

Remarks:	
Date:	Company / Technician signature:
Unit S/N:	

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XO CARE A/S